■ Livella GmbH  
Lokstedter Steindamm 3 b | 22529 Hamburg  
Germany  
info@livella.de | www.livella.de

livella.official livella.uk livella

Version 05, as of 06.2023

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EN

## Silver Nursing Cups to soothe sore breastfeeding nipples

### Content

- 1 Use
- 2 Application time
- 3 Cleaning and care
- 4 Important information

You can wear the silver nursing cups comfortably under your nursing bra and nursing pads. To prevent the cups from slipping, you can fix them in place using medical adhesive tape.

#### 2 Application time

Wear your silver nursing cups before and after breastfeeding, and only take them off during feeding sessions. You can stop using them as soon as your nipples have healed. Should the pain come back, you go back to using the silver nursing cups to protect your nipples.

Silver cleaning products or silver cleaning towels should not be used. These products may contain harmful substances. Detailed care instructions can be found under [www.livella.de/home](http://www.livella.de/home).

#### 4 Important information

- For personal use only.
- Keep out of reach of children. Danger of suffocation, when swallowed.
- Keep away from heat. The silver might pose a burn hazard when heated.
- Do not put your silver nursing cups in the freezer. Frostbite hazard when frozen.
- Do not insert into body orifices. The silver nursing cups are for external use only.
- If you are allergic to silver, do not use the product.

### 1 Use

The Livella Silver Nursing Cups are intended for use between breastfeeding sessions. They cover the sore nipples and protect them from drying out and rubbing against clothing and shield them from external pressure. They further prevent the nipples from getting twisted and from sticking to clothing. The mechanical barrier covers the sore area of the breast and thus supports the body's natural wound healing capabilities.

Clean your silver nursing cups before the first use by boiling them for 20 minutes. Thereafter, we recommend cleaning them on a daily basis with warm water and a mild soap. **Caution: Burn hazard during boiling.**

After breastfeeding, you can place the silver nursing cups directly on your nipples. Adding a few drops of breast milk can support the body's natural wound healing process. Using other care products for sore nipples is neither needed nor recommended.

### 3 Cleaning and care

Tarnishing or „discolouration“ of the silver is a natural process (oxidation), which is caused through exposure to moisture, oxygen and light. However, it has no influence on the use of the product, but rather proves the high purity of the material.

In order to prevent discolouration, the silver nursing cups should be cleaned regularly (as well as after the final use) with a mixture of sodium bicarbonate, salt and water. To do so, line a bowl with aluminum foil and fill it with hot (but not boiling) water. Add 1 tablespoon of baking soda and 1 tablespoon of salt, before soaking the silver nursing cup in the mixture for 1-2 minutes. Important: Do not let them soak for longer than 2 minutes. Take the silver nursing cups out of the mixture, rinse them thoroughly with clear water, dry them and store them in their little pouch in a dark, dry place.

- Stop wearing your silver nursing cups if your skin becomes irritated.
- Do not use the silver nursing cups on open wounds or on inflamed skin (e.g. caused by a fungal, viral or bacterial infection). If symptoms persist, consult a doctor before use.
- Reporting serious incidents: In case of serious incidents in connection with the product, i.e. death or serious deterioration of a person's state of health, the incident must be reported to the manufacturer and the competent authority of the member state in which the user is established.

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